


# SELEKT™

## SELEKT DRENCHER TOOL

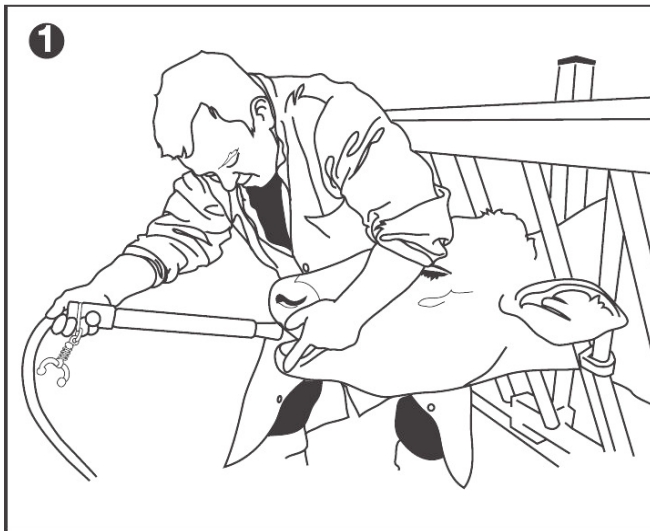

### A. INSERT THE PLASTIC HOSE



Make sure the tip of the plastic hose is drawn completely back to the biting piece and that the nose tong is open.

Hold the corner of the mouth with your free hand (drawing 1) and insert the biting piece at an angle parallel to the tongue until the nose tong can be secured in the nostrils (drawing 2).

Turn the head lightly to the right and push the plastic hose through the biting piece (drawing 3) until app. 1-1½ m hose is left. The tip of the plastic hose is now in the rumen.

Fix the hose by twisting the flat handle on the biting piece.

Pump with even strokes (drawing 4).

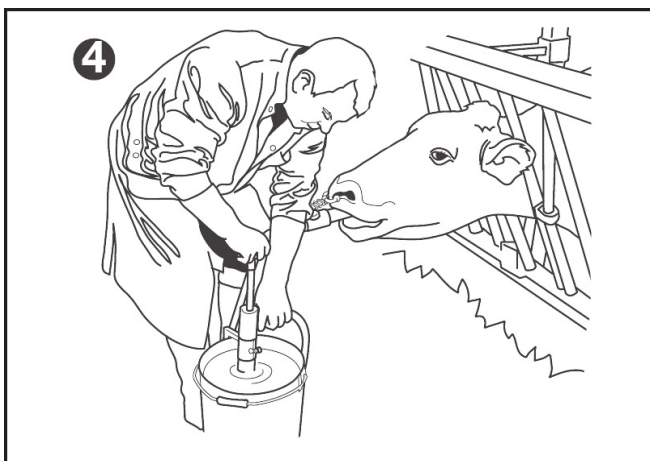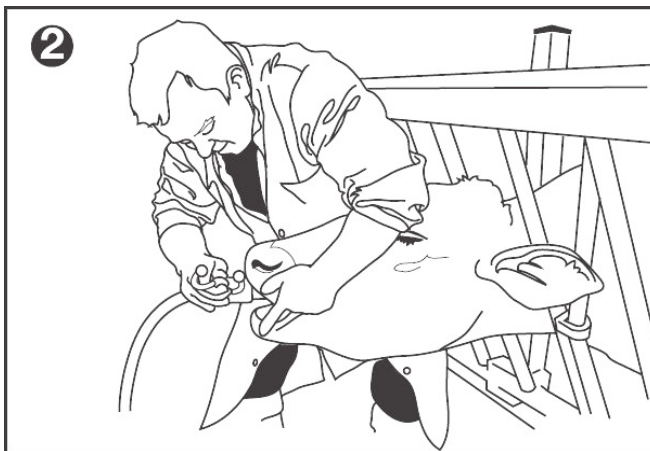

### B. REMOVE THE PLASTIC HOSE


Twist back the handle on the biting piece in order to loosen the hose again.

Make sure to pull the plastic hose all the way back before you open the nose tong.

Pull out the biting piece.

Clean and rinse all part of SELEKT Drencher Tool with warm water (max 50°).

## SELEKT DRENCHER SÆT

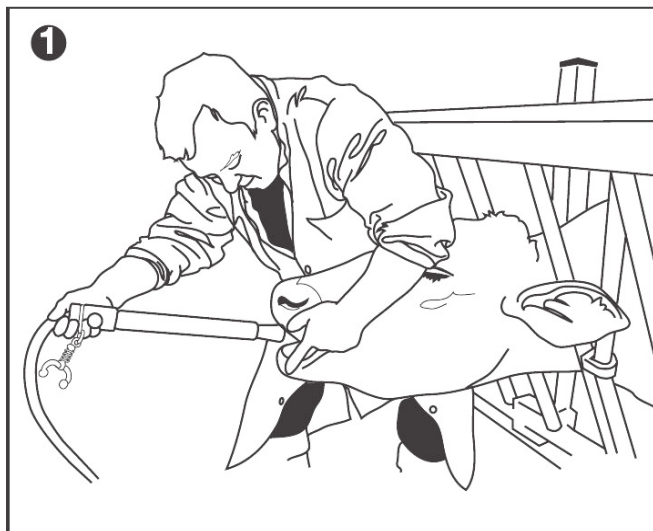


### A. INDFØRING AF PLASTIKSLANGE

Sørg for at plastikslangens hoved er trukket helt tilbage til bidestykket og at næsebremsen er åbnet.



Grib mundvigen med den frie hånd (billede 1) og før det hvide bidestykke ind parallelt med tungen indtil næsebremsen kan anbringes i næseborene (billede 2).

Drej hovedet let til højre og skub plastikslangen gennem bidestykket (billede 3) indtil der er ca. 1-1 ½ m slange tilbage. Plastikslangens hoved ligger nu i vommen.

Spænd knappen på bidestykkets håndtag, så slangen sidder fast.

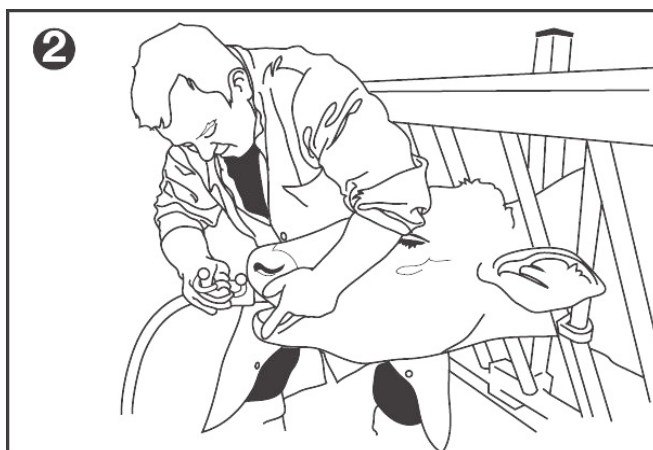

Pump med jævne træk (billede 4).

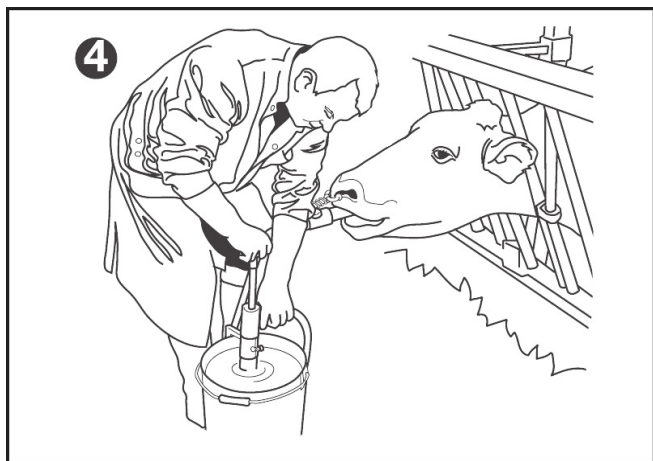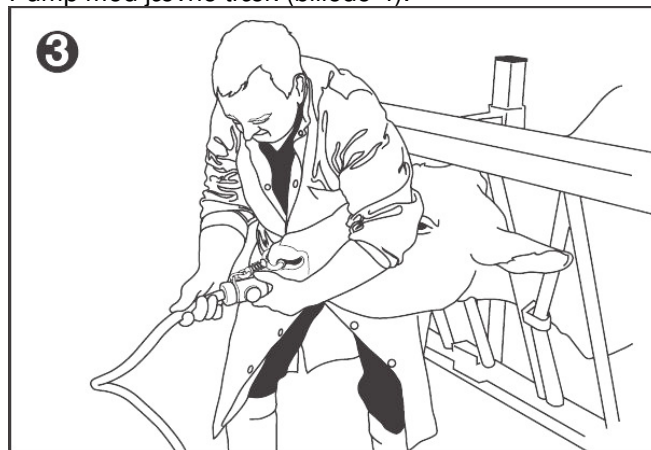

### B. UDTAGNING AF PLASTIKSLANGE

Løsn knappen igen.

Træk plastikslangen helt tilbage før næsebremsen åbnes.

Træk bidestykket ud.

Rens og skyl alle dele af pumpeværket med varmt vand tilsat desinfektionsmiddel.